

SleepWeaver® Anew™ Online Sizing Template

Instructions for Use:

1. To ensure the Sizing Guide is true to scale, verify the printer is set to print at 100%.
2. After printing, check the size of the printed version of the Sizing Guide by lining up a ruler with the ruler scale on the guide.
3. Carefully cut out the Sizing Guide on the dotted line.
4. Remember, a soft fabric mask is different than a hard plastic mask. Fabric can be pulled tighter and relaxes as it is used daily.
5. If you fall "in between" sizes, please order the smaller size of Anew.

SleepWeaver®
Anew™

Sizing Guide

B : Bridge of Nose

4 in

3 in

2 in

1 in

0 in

LARGE 100965 ↗

REGULAR 100960 ↗

SMALL 100955 ↗

	Small	Regular	Large
Nasal Width	1 in 2.5 cm	1.25 in 3.2 cm	1.5 in 3.8 cm
Height	3.25 in 8.2 cm	3.5 in 8.9 cm	3.75 in 9.5 cm

Start Height Measurement from the Middle of Your Chin to the Bridge of Your Nose

Measure Nasal Width

A : Middle of Chin ↗

888-825-9640
www.circadiance.com