Transcend AUTO™ Quick Guide

ENGLISH

FRANÇAIS

DEUTSCH

ITALIANO

ESPAÑOL

NEDERLANDS

PORTUGUÊS

TRANSCEND INTRODUCTION

Thank you for choosing Transcend AUTO, the first auto-titrating obstructive sleep apnea therapy device designed to fit your active lifestyle. Transcend Auto is a fully-featured APAP that continually monitors your breathing to adapt to your changing therapy needs. This device provides pressure relief on exhalation with EZEX pressure relief technology. Weighing less than a pound (426 g) Transcend AUTO is the most innovative small, light and portable APAP system, truly giving you the freedom to sleep anywhere.

R_xonly

Begin by inspecting all items shipped to you. Please report any damaged or missing items to the home healthcare provider where you obtained your device.

Note: For more detailed device descriptions, cautions and troubleshooting procedures refer to the Transcend AUTO User Manual that is available to download at www.mytranscend.com/resources or by calling Somnetics customer service at 877-621-9626 to request a copy.

TRANSCEND COMPONENTS

Included with the Transcend AUTO:

- | | | | |
|---|--|---|-----------------------------------|
| 1 | Transcend AUTO | 5 | Travel Bag |
| 2 | Standard 6 foot (183 cm) hose – not HME compatible | 6 | CD containing user manuals |
| 3 | Universal hose adaptor | 7 | CD containing compliance software |
| 4 | Multi-plug Universal AC Power Supply Set (PSA2) | 8 | Quick Guide |
| | | 9 | USB cable |

What's Not Included (optional accessories sold separately):

- Transcend Heated Humidifier™
- Transcend Base Station (with LCD display or Basic)
- Transcend P₄ and P₈ Battery Systems
- Transcend Portable Solar Charger
- Transcend Mobile Power Adaptor

OPTIONAL ACCESSORIES

TRANSCEND HEATED HUMIDIFIER™

The Transcend Heated Humidifier provides premium comfort at home. The water-based humidifier integrates completely with Transcend to provide heated humidification in one small footprint. Adjustable heat settings provide user control to reach the optimum level of comfort during therapy.

- Sleek, integrated design
- Small footprint
- Adjustable heat settings
- Warm up indicator
- Dishwasher-safe water reservoir
- Compatible with any mask
- Compatible with Transcend, Transcend EZEX and Transcend Auto

TRANSCEND PORTABLE BATTERY OPTIONS

Transcend P₄ Battery: Weighs 0.6 pounds (272 g).

Charge with the Transcend Multi-plug Universal Power Supply, Mobile Power Adaptor, or Portable Solar Charger. The P₄ Battery provides approximately 7-9 hours therapy at a fixed pressure of 14 cmH₂O.*

Transcend P₈ Battery: Weighs 1.1 pounds (499 g).

Charge with the Transcend Multi-plug Universal Power Supply, Mobile Power Adaptor, or Portable Solar Charger. The P₈ Battery provides approximately 14-16 hours therapy at a fixed pressure of 14 cmH₂O.*

* Actual battery performance may vary depending upon altitude, leak, respiratory rate and tidal volume.

OPTIONAL ACCESSORIES

TRANSCEND DC MOBILE POWER ADAPTOR

The DC Mobile Power Adaptor lets you use Transcend in your vehicle, RV, or boat. The Mobile Power Adaptor may also be used to recharge your Transcend batteries when wall power is unavailable.

TRANSCEND PORTABLE SOLAR BATTERY CHARGER™

The Portable Solar Battery Charger uses solar power to recharge your Transcend battery. Solar modules begin charging immediately when placed in the sun, even in low light conditions.* Rolled up, it's about the size of a folded newspaper and easily stows for your next outdoor adventure.

* Battery recharge times vary depending upon direct sun conditions.

TRANSCEND BASE STATIONS

Transcend Base Station: The Transcend Base Station provides extra stability as a bedside anchor for the Transcend Auto. While not required for home use of the Transcend Auto, in some cases use of the Base Station may be preferred.

Transcend LCD Programmable Base Station:



In addition to providing enhanced bedside stability, the Transcend LCD Programmable Base Station allows users to view basic therapy compliance information and adjust ramp settings. The Programmable Base Station may also be used by the clinician to change therapy pressure when used in the Clinician Mode (Clinician Mode access is not available to patient users).

ASSEMBLING YOUR TRANSCEND AUTO

To completely assemble the Transcend Auto:

1. Attach the Universal Hose Adaptor and 6-foot hose to the APAP.
2. Connect the interface (mask) to the patient (open) end of the 6-foot hose.
3. Plug the power supply barrel connector into the APAP with the cord pointed up.
4. Connect the power supply to a power source.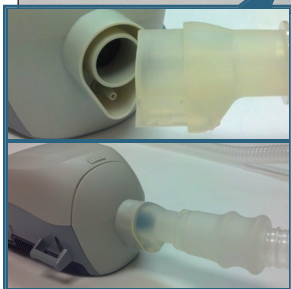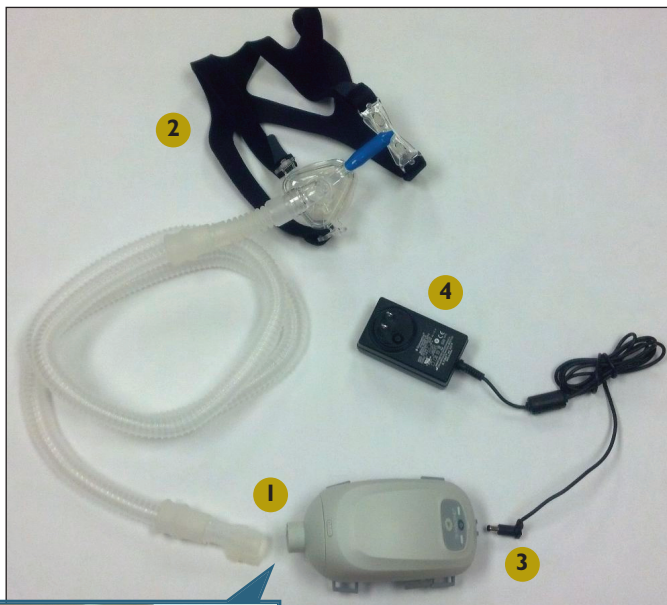

Note: Your Transcend AUTO can also be powered by the optional Transcend Mobile Power Adaptor or Transcend Battery Systems.

Note: When power is supplied to the APAP, the yellow light will flash once followed by three quick flashes of the green LED light. This sequence indicated power is being supplied to the device and that it has successfully entered Standby mode.

Note: Do not use a power converter with your Transcend as it may damage the device. Transcend automatically converts voltage.

TO CHANGE THE PLUG ADAPTOR ON THE MULTI-PLUG UNIVERSAL AC POWER SUPPLY:


1. Depress the button on the power supply and rotate the plug adaptor counterclockwise to unlock.
2. Line up the new plug adaptor into the power supply and turn clockwise until the plug snaps into place.

STARTING AND ENDING THERAPY

Starting Therapy

1. The Transcend control panel consists of a Power button (bottom), Ramp button (top), and two LED lights. (Refer to image)
2. Ensure that the device is connected to a power source and is in Standby mode.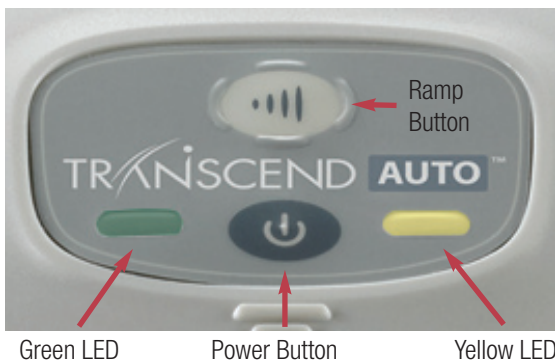

3. Put on your mask and make sure that proper fit is achieved.
4. Press the lower, gray Power button () to turn on Transcend and enter Therapy mode. Air flow will begin as the blower delivers your prescribed therapy pressure.

Using the Ramp Function

Ramp settings will be programmed by your healthcare provider as prescribed.

1. Initiate ramp by pressing the Ramp (top) button after the Power button has been pressed.

Note: The Ramp button on the Transcend will not work unless your healthcare provider has preset the ramp function.

2. Pressure from the device will be lowered to a preset starting pressure and will gradually increase over a preset time interval until the device reaches your prescribed starting therapy pressure.
3. To accelerate the ramp time, hold down the Ramp button. The device will quickly increase pressure until reaching your prescribed therapy pressure.

Ending Therapy

To end therapy, press the Power button to return the device to Standby mode.

CARING FOR YOUR TRANSCEND AUTO AND COMPONENTS

Drying Mode

Drying mode should be used daily to dry the inside of your Transcend after use.

Caution: Do not attempt to use Transcend while in Drying mode. The device will not provide prescribed therapy pressure while in Drying mode.

1. Set the device on a dry, stable surface. A cloth surface is not recommended as it may get soiled.
2. Make sure the device is in Standby mode.
3. To enable Drying mode, press and hold the Ramp button and then press the Power button.
4. Low pressure airflow will begin and last for 30 minutes until the drying sequence is completed. The device will then return to Standby mode. To end drying mode before the 30 minute sequence is completed, press the Power button. The device will return to Standby mode.

Caution: Drying mode will dry the inside of your Transcend. Do not attempt to clean the inside of the Transcend device.

Cleaning the Exterior

Clean the exterior of your Transcend device as needed. To clean, wipe the surface with a lint-free cloth or paper towel that has been dampened in water containing mild dishwashing detergent*. Following cleaning, wipe again with a cloth or paper towel moistened with clear water. Allow to dry.

Caution: Never submerge Transcend in water. Do not allow soap or water to enter the device.

Cleaning the Transcend Components

The interface (mask) should be washed weekly. The air supply tube (hose) should be washed daily.

1. Completely disconnect the components.
2. Hand wash in warm water with a mild dishwashing detergent*.
3. Rinse thoroughly.
4. Allow to air dry. Components should be completely dry before reconnecting.
5. Reconnect the components and reattach to Transcend.



Caution: Do not put any Transcend components in the dishwasher or laundry machine.

* A solution of 5% mild liquid detergent in distilled water (1.6 fl oz liquid detergent per quart of distilled water) is recommended.

CLEANING AND REPLACING THE FILTER MEDIA

The Transcend AUTO Filter Media should be replaced every six months. Contact your homeware dealer to purchase a replacement Filter Media.

1. Remove the filter frame by gently pressing the button release and lifting the plate away from the APAP (Image 1).
2. Remove the Filter Media from the filter frame and discard.
3. Insert a new Filter Media into the filter frame (Image 2).
4. To replace the new filter frame, lay it on the bottom of the APAP. Snap the front clips into place first, then snap the connector into place.

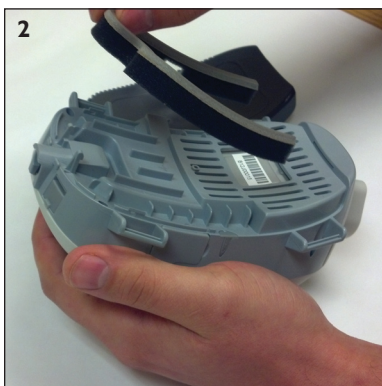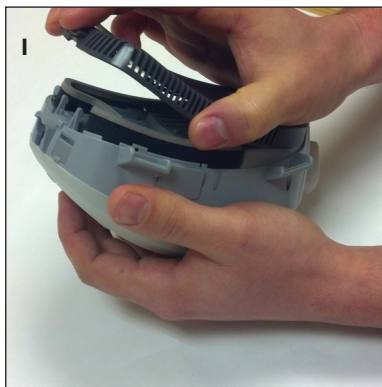The Filter Media should be removed and cleaned weekly.

1. To clean the Filter Media, first submerge a lint-free cotton cloth into the cleaning solution*.
2. Wring excess water from the cloth.
3. Gently pat the Filter Media with the cloth.
4. Dry surfaces by wiping with a dry, lint-free cotton cloth.
5. Allow the Filter Media to air dry at room temperature for approximately 30-60 minutes before reassembly. Drying time will depend on ambient conditions.
6. Reconnect the Filter Media and filter frame to the Transcend AUTO.

Note: Earlier versions of Transcend may require that the complete filter frame be replaced. Refer to the User Manual supplied with the device.

* A solution of 5% mild liquid detergent in distilled water (1.6 fl oz liquid detergent per quart of distilled water) is recommended.

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©2014 Somnetics International, Inc. All rights reserved.