



PHILIPS

RESPIRONICS

DreamWear

Under the nose
nasal mask

Rediscover
dreams

Like wearing nothing at all

Patients can sleep how they want with the freedom and comfort of DreamWear

Patients can sleep like it's not even there

Do your patients have trouble finding a comfortable sleeping position or feel restricted by CPAP tubing? DreamWear is an innovative mask created to give your patients the choice to sleep how they want.

The best of pillows and nasal masks


DreamWear's unique design offers many of the benefits of nasal and pillows masks to allow patients to have the best of both mask types. Allowing more freedom of movement and more comfort than their prescribed mask, DreamWear patients feel like they are not wearing a mask at all.*



Benefits

- Tubing attachment located at the top of the head to offer freedom of movement during the night
- Prevents red marks, discomfort or irritation in the nostrils or on the nose bridge**
- Users rated DreamWear more stable, more comfortable, and more appealing than their prescribed mask*
- Soft, flexible silicone frame and fabric wraps provide a soft touch on the patient's cheek

Users told us that DreamWear gave them more freedom to move, had a better fit than their prescribed mask, and comes closest to making them feel like they don't have anything on their face during therapy.*

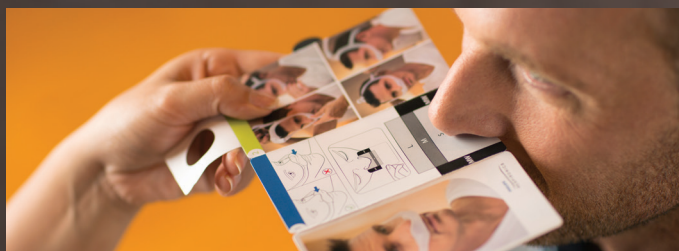

DreamWear cushion size is determined by measuring the length and width of the nose using the simple sizing gauge.**

*2015 Philips User Preference Questionnaire, data on file.

**Mask does not directly contact the bridge of the nose or nostrils

***Medium wide (MW) cushion size uses length and width of the nose



Get started

To learn more about why DreamWear can be one of the only nasal masks you offer your patients, contact your Philips Respironics representative for more information.

DreamWear fitpak	Part number
-------------------------	--------------------

DreamWear with headgear, medium frame, all cushion sizes	1116700
----------------------------------------------------------	---------

DreamWear without headgear, medium frame, all cushion sizes	1116701
-------------------------------------------------------------	---------

DreamWear with headgear, small frame	Part number
---------------------------------------------	--------------------

Small cushion	1116685
---------------	---------

Medium cushion	1116686
----------------	---------

Large cushion	1116687
---------------	---------

Medium wide cushion	1116688
---------------------	---------

DreamWear with headgear, medium frame	Part number
----------------------------------------------	--------------------

Small cushion	1116680
---------------	---------

Medium cushion	1116681
----------------	---------

Large cushion	1116682
---------------	---------

Medium wide cushion	1116683
---------------------	---------

DreamWear with headgear, large frame	Part number
---------------------------------------------	--------------------

Small cushion	1116690
---------------	---------

Medium cushion	1116691
----------------	---------

Large cushion	1116692
---------------	---------

Medium wide cushion	1116693
---------------------	---------

DreamWear without headgear, small frame	Part number
------------------------------------------------	--------------------

Small cushion	1116710
---------------	---------

Medium cushion	1116711
----------------	---------

Large cushion	1116712
---------------	---------

Medium wide cushion	1116713
---------------------	---------

DreamWear without headgear, medium frame	Part number
-------------------------------------------------	--------------------

Small cushion	1116705
---------------	---------

Medium cushion	1116706
----------------	---------

Large cushion	1116707
---------------	---------

Medium wide cushion	1116708
---------------------	---------

DreamWear without headgear, large frame	Part number
------------------------------------------------	--------------------

Small cushion	1116715
---------------	---------

Medium cushion	1116716
----------------	---------

Large cushion	1116717
---------------	---------

Medium wide cushion	1116718
---------------------	---------

DreamWear replacement parts	Part number
------------------------------------	--------------------

Small nasal cushion	1116740
---------------------	---------

Medium nasal cushion	1116741
----------------------	---------

Large nasal cushion	1116742
---------------------	---------

Medium wide nasal cushion	1116743
---------------------------	---------

Headgear	1116750
----------	---------

Small frame	1116745
-------------	---------

Medium frame	1116746
--------------	---------

Large frame	1116747
-------------	---------


Fabric wraps	1116754
--------------	---------




DreamWear patient fitting guide


Achieving the right fit with DreamWear is easy. Simply follow the instructions below to get started.


Four cushion sizes (Small, medium, large, medium wide) are available. Please read instructions for use for further fitting information.

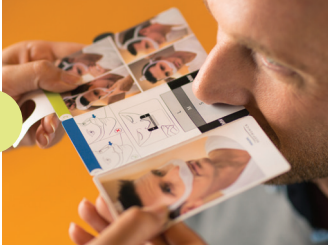- 

1 Hold the sizing gauge horizontally under the nose and select the cushion size based on where the outer edge of the nostrils and tip of the nose contact the gauge.
- 

2 The cushion opening will be directly under the nostrils. The cushion will hug the nose and leak will be minimal.
(At no time should the nose be inside the cushion opening. If any part of the cushion overlaps the nose, the cushion is being worn incorrectly.)
- 

3 With the mask assembled, place the cushion under the nose.
- 

4 Position the frame on the top of the head.
- 

5 Pull the headgear over the back of the head.
- 

6 To adjust the mask, peel the headgear tabs away from the fabric. Adjust the straps length. Press the tabs back against the fabric to reattach.
(Note: Do not overtighten the headgear.)

7



Position the mask until it fits comfortably. When finished, the elbow should rest at the top of the head.

Mask frame sizing

The medium (MED) mask frame will comfortably fit most faces. If the MED frame does not fit a patient's face, a small (SM) or large (LG) mask frame might better suit your patient's needs. **Call for more information, 1-800-345-6443.**

Part of the **Dream Family**

DreamWear is part of the Dream Family from Philips Respironics. The Dream Family offers innovative, comprehensive sleep therapy technology like our sleep therapy system, DreamStation, and our patient self-management tool, DreamMapper.



Call for more information, 1-800-345-6443.



©2015 Koninklijke Philips N.V. All rights reserved.
Specifications are subject to change without notice.
Trademarks are the property of Koninklijke Philips N.V.
(Royal Philips) or their respective owners.

www.philips.com

Caution: US federal law restricts these devices to
sale by or on the order of a physician.

RRDPGH EB 9/10/15 MCI 4106842 PN 1125118
1010 Murry Ridge Lane, Murrysville, PA 15668
800-345-6443 • 724-387-4000