RESMED

Mirage Activa™ LT

NASAL MASK

Fitting Guide



• Turn the MicroFit dial to position 24, so that the forehead support is fully extended.



- · Disconnect the lower headgear clip, hold the mask over the nose and pull the headgear over the head.
- Reconnect the lower headgear clip ensuring it clicks into place.



· Keep the bottom straps loose to allow room for the cushion to inflate. Use two fingers as a guide.



· Connect one end of the air tubing to the swivel and the other end to the airflow device.



· Turn on the airflow device.



While lying down, adjust the dial for comfort and seal.



· If the mask is fitted correctly, the cushion should be smooth and fully inflated.



• If fitted incorrectly, the cushion will be creased and underinflated.



For details, refer to the Mirage Activa LT Nasal Mask User Guide.

ResMed Corp Poway, CA, USA +1 858 746 2400 or 1 800 424 0737 (toll free), ResMed Ltd Bella Vista, NSW, Australia +61 (2) 8884 1000 or 1 800 658 189 (toll free). Offices in Austria, Belgium, Brazil, China, Finland, France, Germany, Greece, Hong Kong, India, Ireland, Italy, Japan, Malaysia, Mexico, Netherlands, New Zealand, Norway, Portugal, Singapore, South Africa, Spain, Sweden, Switzerland, United Kingdom (see website for details). Active, ActiveCell, and Mirage are trademarks of ResMed Ltd. and registered in U.S. Patent and Trademark Office. ©2009 ResMed. 1013306/2 09 01