

## Sizing Guide

SleepWeaver Advance Masks				
	Advance Small	Between*	Advance	Measurement Technique
<b>Nasal Width</b>	Less than 1 1/4 inches	1 1/4 inches to 1 1/2 inches	Greater than 1 1/2 inches	1.) Measure from outer left nare to outer right nare. (see diagram below)

\*If the measurement is between 1 1/4" and 1 1/2" then the patient should try the Small first. Users with narrow and/or flat nose bridges should also try the Small first.

- 1.) The current Advance may be necessary if the patient has a longer nose
- 2.) The use of this information is not intended to replace the advise of a medical professional, sleep technician, or HME Dealer.
- 3.) Remember, a soft fabric mask is different than a hard plastic mask. Pull mask snug, not tight.

