

## Fitting Guide for AirFit P10




- Hold the pillows at your nose, ensuring that the pillow size indicator is on the bottom and you can see the left (L) and right (R) orientation indicators.



- Stretch the lower strap around the back of your head.
- The top strap should sit comfortably on top of your head.



- Pull the headgear over your head by holding the lower strap.

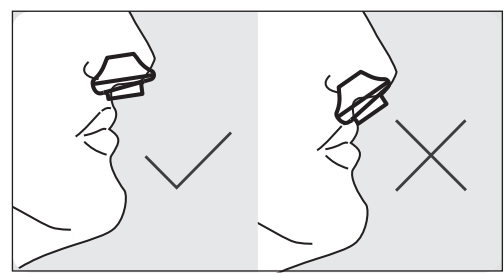

- Move the mask away from your face, and then place it back again.
- Ensure the nasal pillows fit securely into your nostrils with both pillows positioned upright.



- Finally, connect the end of the mask's tube into your device's air tubing.
- Turn on your device — your mask is now ready to use.



- If adjustment is needed, simply spread the upper and lower straps apart to loosen, or draw them closer together to tighten.

**Note:** For complete user instructions, refer to the AirFit P10 User Guide.

**Rx Only**

\*Declared dual numbered noise emission values (with uncertainty 3dBA) in accordance with ISO 4871.

<sup>1</sup> ResMed Testing Ref: D000-0201; eLB0798-206. <sup>2</sup> ResMed Clinical Study comparing AirFit P10 to Swift FX Ref: D629-117. ResMed objective data (pressure, leak, usage, AHI).

ResMed Corp San Diego, CA, USA +1 858 836 5000 or 1 800 424 0737 (toll free). ResMed Ltd Bella Vista, NSW, Australia +61 (2) 8884 1000 or 1 800 658 189 (toll free). See ResMed.com for other ResMed locations worldwide. AirFit, QuickFit and QuietAir are trademarks of ResMed Ltd. Swift is a trademark of ResMed Ltd and is registered in the U.S. Patent and Trademark Office. ©2013 ResMed. Specifications may change without notice. 1016801/3 2013-12



## A quieter mask to fit your sleep.

The AirFit™ P10 nasal pillows mask is part of ResMed's new AirFit series—the right fit for you. This is our quietest mask yet, offering the reliability and performance of the Swift™ FX in a design that's approximately 50% lighter and 50% quieter.<sup>1</sup> A study also found that patients using the AirFit P10 slept more than 40 minutes longer per night.<sup>2</sup>

### Quieter

QuietAir™ woven-mesh vent technology makes this ResMed's quietest mask yet, promoting patient acceptance.

### Lighter

50% lighter than our most popular nasal pillows mask,<sup>1</sup> offering patients the comfort of a better night's sleep.

### Easier to Use

Key-clip nasal pillows feature color-coded sizing and left/right orientation for fast and easy fitting.



# More than 40 minutes of extra sleep, every night

In a recent clinical study, we found that patients slept more than 40 minutes longer per night when using the ResMed AirFit P10.<sup>2</sup> That's almost five hours per week of additional sleep, helping ensure your patients will stay compliant with their sleep therapy.




## AirFit P10 Features and Benefits

**Dual-wall technology** with trampoline action seals on contact, dispersing air gently to the nose.

**Key-clip nasal pillows** offer effortless assembly, with color coding for easy sizing and left/right orientation to ensure patients fit the mask properly.

**Innovative flexible tube** is ultra light and flexible yet highly durable, offering patients full freedom of movement without compromising seal.

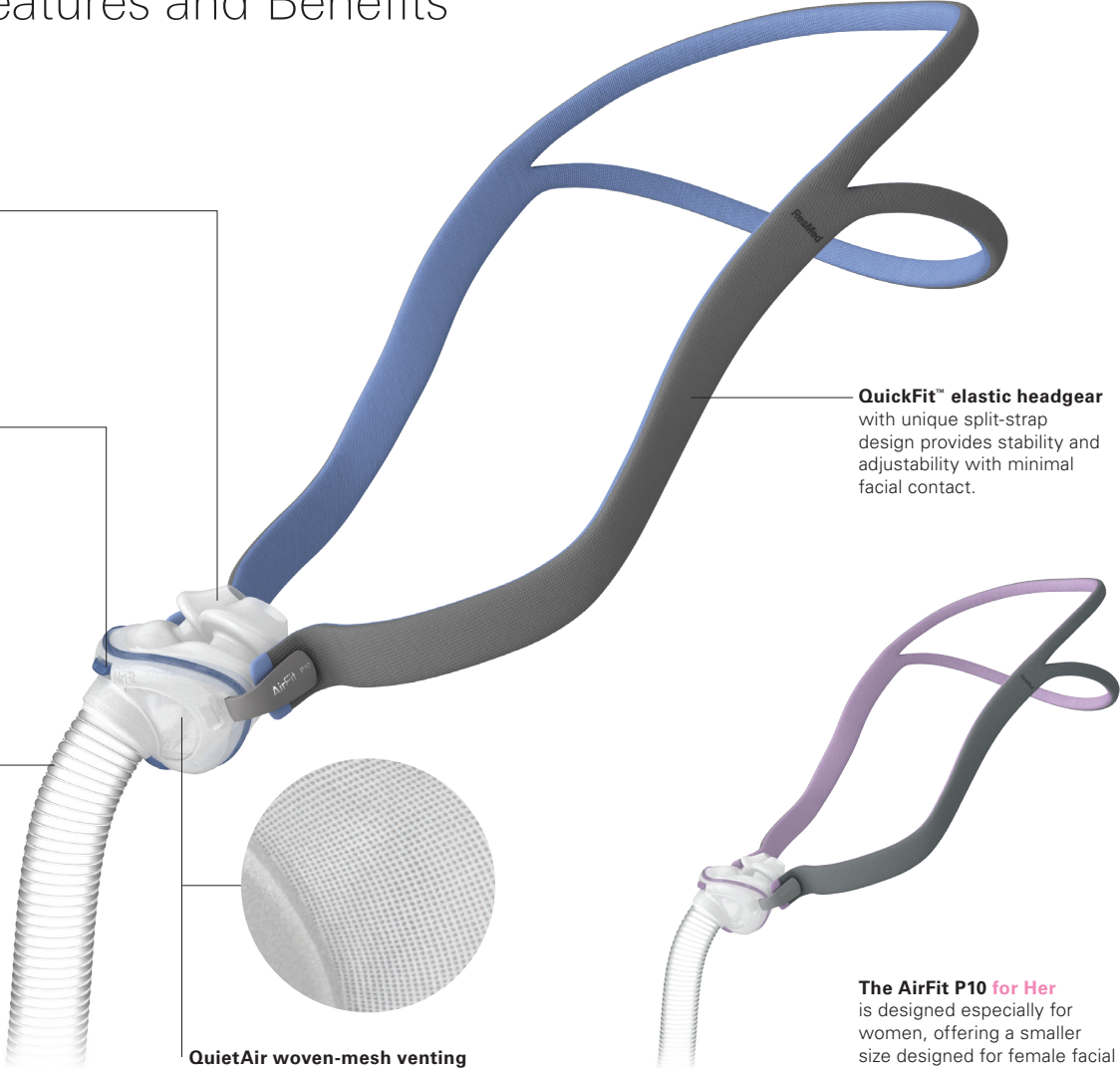


**QuietAir woven-mesh venting** quietly directs exhaled air, so it's barely noticeable to patients and their bed partners.

**QuickFit™ elastic headgear** with unique split-strap design provides stability and adjustability with minimal facial contact.

**The AirFit P10 for Her** is designed especially for women, offering a smaller size designed for female facial contours, along with stylish tones and highlights.




## AirFit P10 Components

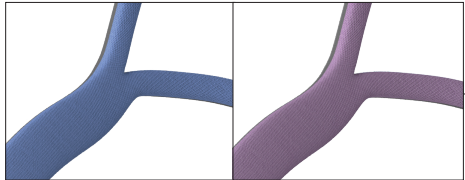

QuickFit elastic headgear  
**62935**

QuickFit for Her elastic headgear  
**62936**

Frame system (includes frame and pillows)  
**62920 (XS)**  
**62921 (S)**  
**62922 (M)**  
**62923 (L)**

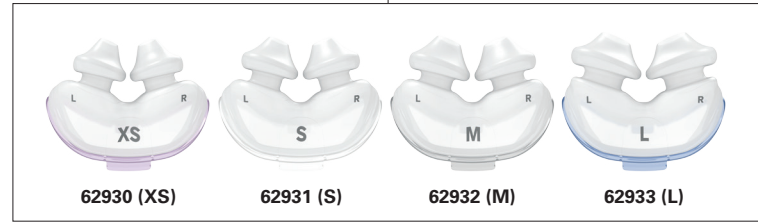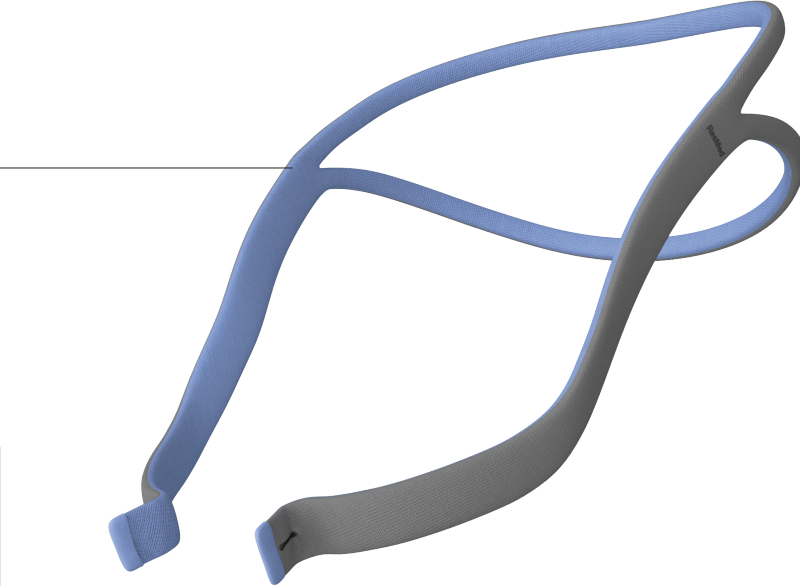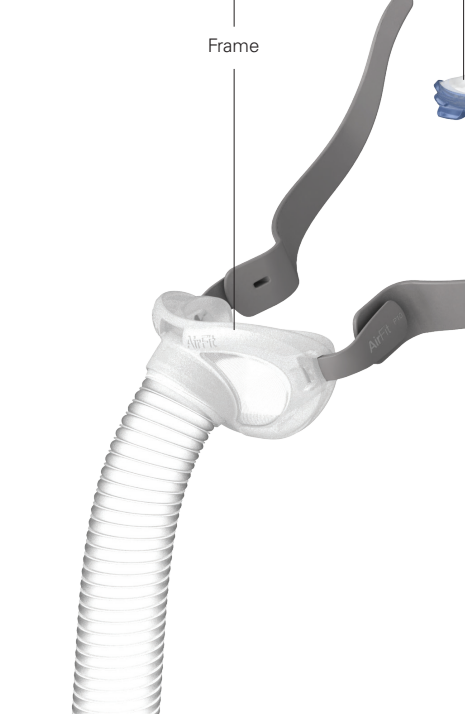

**62930 (XS)**   **62931 (S)**   **62932 (M)**   **62933 (L)**

Pillows

**3-part simplicity** makes the AirFit P10 50% lighter than the leading nasal pillows mask.<sup>1</sup>

- XS** Extra Small
- S** Small
- M** Medium
- L** Large

### Product Codes

AirFit P10 - complete system*	<b>62900</b>
AirFit P10 for Her - complete system*	<b>62910</b>

\* S, M and L pillows come packaged with the mask.  
\*\* XS, S and M pillows come packaged with the mask.

**Note:** For details, refer to the AirFit P10 User Guide.

<sup>1</sup>Combined responses of those expressing a preference in an independent patient survey in the USA, UK, Germany, France & Australia, from September 2012. Visit [www.resmed.com/maskbrand](http://www.resmed.com/maskbrand).

